



sagemax[®]


Multilayer Concept

NexxZr^{®+} Multi 2.0

Incisal **3 mm**

Mixture **4 mm**

Dentin
7, 9 or 13 mm
depending on
disc thickness

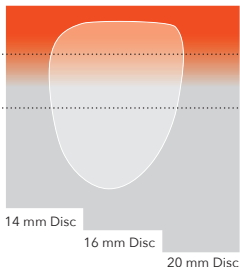

650 MPa
49 % Translucency

Average Values

850 MPa
45 % Translucency


i The height positioning in the disc determines the amount of incisal area in the restoration.



sagemax®

Standard Sintering Program

NexxZr®+ Multi 2.0

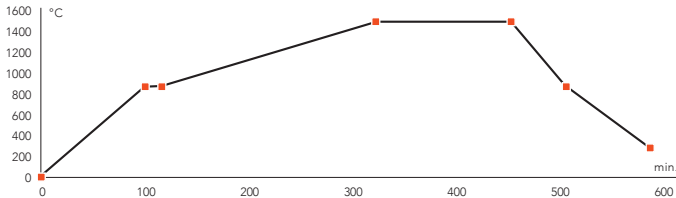

Phase	Temperature °C	Heating/Cooling rate °C/min.	Temperature °F	Heating/Cooling rate °F/min.	Holding time min.
1	20 – 900	10	68 – 1652	18	30
2	900 – 1500	3.3	1652 – 2732	5.94	120
3	1500 – 900	10	2732 – 1652	18	-
4	900 – 300	8.3	1652 – 572	14.94	-



Sagemax Bioceramics, Inc., 34210 9th Ave. South, Suite 118
Federal Way, WA 98003, USA



See instructions **CE 0123** Rx only. For dental use only!



sagemax.com