

sagemax.com

sagemax®



Sagemax experience

NexxZr T Multi
facial cut-back
by Mark Bultmann

NexxZr T Multi facial cut-back
by Mark Bultmann

Sagemax experience

Fabrication and rework



Fabrication of the NexxZr T Multi restoration.



Manual finishing of the fissures with a square burr.

Light blocking



BriegelDental's opaquer liquid LightBlock is used to opacify the zirconium oxide at the bonding face.



Shake liquid and activator and mix in equal parts, ratio 1:1.



Apply LightBlock liquid to the inside of the restoration.

Sintering



Restoration after sintering.



Perfect sintering result. Excellent fit. Outstanding shade and translucency gradient.

Facial cut-back



Internal staining and wash firing using "sprinkle technique".



Result after wash firing.




Finished facial layering with ZI-CT from Creation.



Bonding between restoration and titanium base with Multilink Hybrid Abutment from Ivoclar.



Finished bonded restoration.

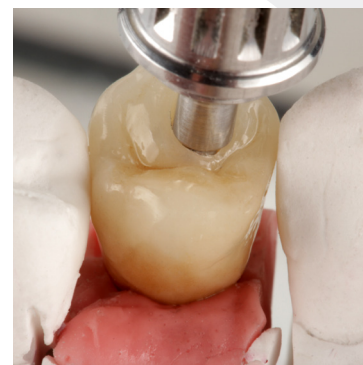

Fit check on the model.

Final result



Finished restoration from occlusal.



Finished restoration from vestibular.



Finished restoration, ready for insertion.



Esthetic outcome in the oral cavity.

Dental designer



Mark Bultmann
Master Dental Technician, Germany

www.viadenta.com

in Mark Bultmann

@ zahntechnik26789

Contact us
info@sagemax.com

sagemax.com

Follow us
[@sagemax.official](https://www.instagram.com/sagemax.official)

