

sagemax.com

sagemax[®]



Sagemax experience

NexxZr T Multi
facial cut-back
by Mark Bultmann


NexxZr T Multi facial cut-back
by Mark Bultmann

Sagemax experience


Initial situation, temporary, color determination & framework fabrication




Initial situation of the patient.

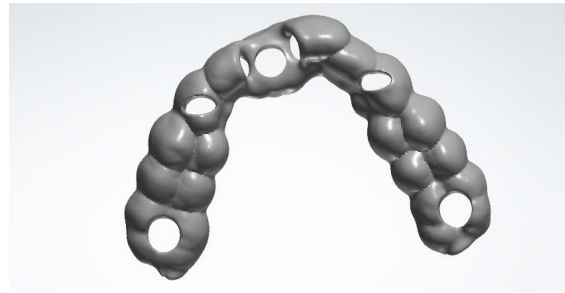


Fabrication of a drilling template.

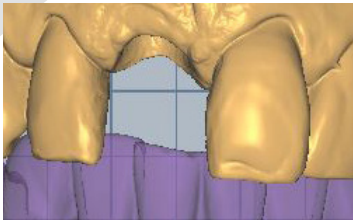

Design of the emergent profile.

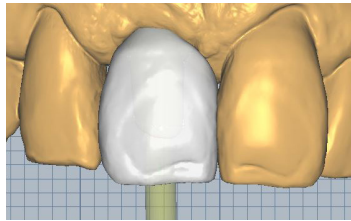

Mirroring of the neighboring tooth.

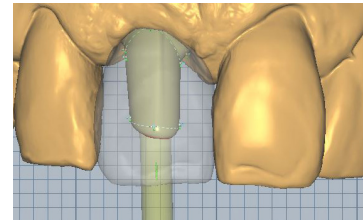


Checking the space conditions.




Temporary crown, after insertion of the implant.



Color determination with the help of a polarizing filter.

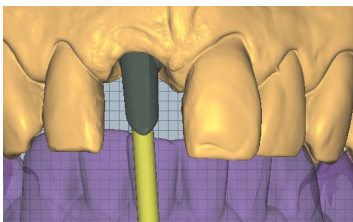

Construction of the abutment.

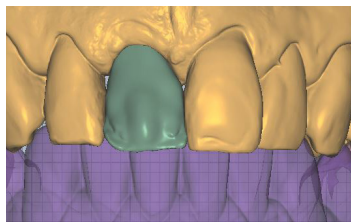

Construction of the framework.



Framework made of NexxZr T Multi.

Conditioning, facial cut-back & finishing




Framework with individually milled abutment.



Surface conditioning of the bonding surface with NexxZr Glaze Spray.

Al₂O₃ blasted surface.

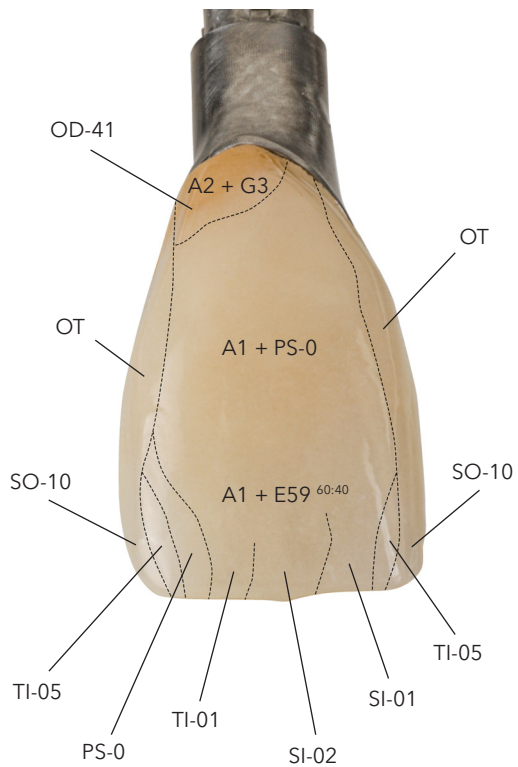

Facial cut-back with Creation ZI-CT (Co. Creation Willi Geller).

Ready-bonded crown on the abutment.

Final result



Before.



After.



Finished restoration, after insertion.



Esthetic final result in the oral cavity.

Dental designers



Mark Bultmann
Master Dental Technician, Germany

www.viadenta.com
in Mark Bultmann
@ zahntechnik26789

Dr. Ulrika Montén
Dr.Dr. Dieter Edinger
Dentists, Germany

www.pro-dental-care.de

Contact us
info@sagemax.com

sagemax.com

Follow us
[@sagemax.official](https://www.instagram.com/sagemax.official)
f in @