

sagemax.com

sagemax®



Sagemax experience

NexxZr T Multi
stained vs layered (cut-back)
by Ivan Unfeter

NexxZr T Multi stained vs layered (cut-back)
by Ivan Unfeter


Sagemax experience

Staining scheme

Material
IPS Ivocolor, Co. Ivoclar


Step 1

Color/Mixture
SD1 + E02 Cream +
Glaze Fluo
Ratio
2/1/2

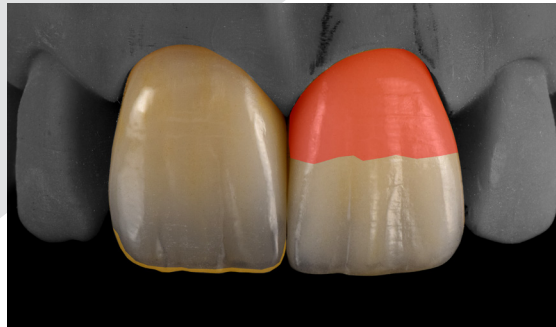


Step 2

Color/Mixture
E15 Ocean + E18 Black
+ Glaze Fluo
Ratio
1/1/1

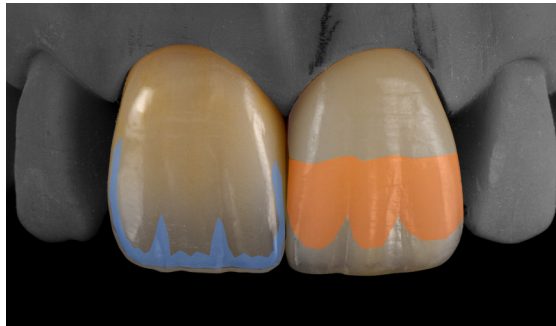


Step 3

Color/Mixture
E02 Cream + SD1 +
Glaze Fluo
Ratio
1/1/1

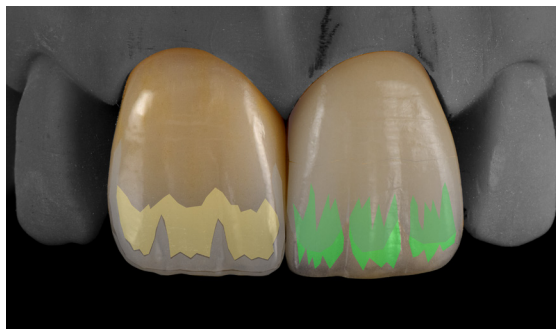

Step 4

Color/Mixture
SD1 + Glaze Fluo
Ratio
1/1

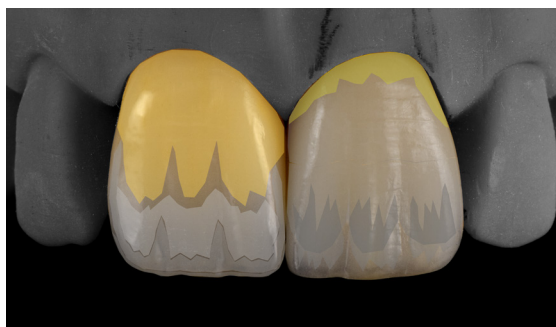

Layering scheme

Material
IPS e.max Ceram, Co. Ivoclar

Step 1

Color/Mixture
Power Dentin A1
Ratio
Pure

Step 2

Color/Mixture
Dentin A1
Ratio
Pure

Step 3

Color/Mixture
Dentin A1 +
Mamelon light
Ratio
1/1

🔥 Dentin firing

Step 4

Color/Mixture
Honey
Ratio
Pure

Step 5

Color/Mixture
E01 White + Glaze Fluo
Ratio
1/1

Stain firing 🔥




Step 5

Color/Mixture
OE 2
Ratio
Pure

Step 6

Color/Mixture
Glaze Fluo
Ratio
Pure

Glaze firing 🔥

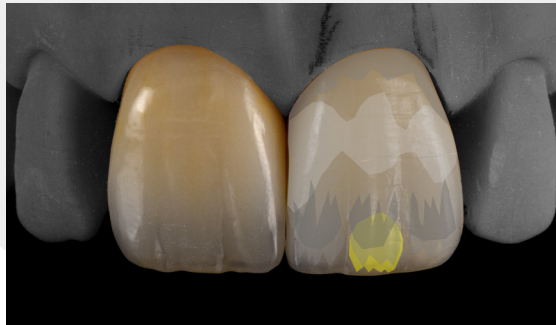


Step 6

Color/Mixture
Citrine
Ratio
Pure

Step 7


Color/Mixture
OE 1
Ratio
Pure

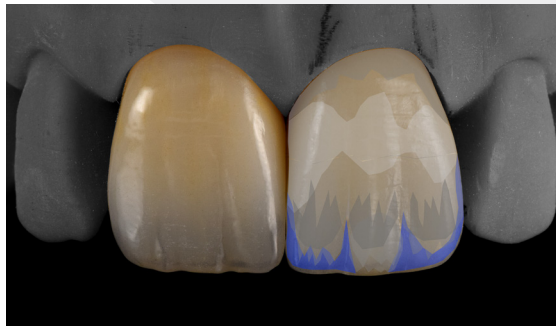

Step 8

Color/Mixture
Dentin A1
Ratio
Pure


🔥 Enamel firing

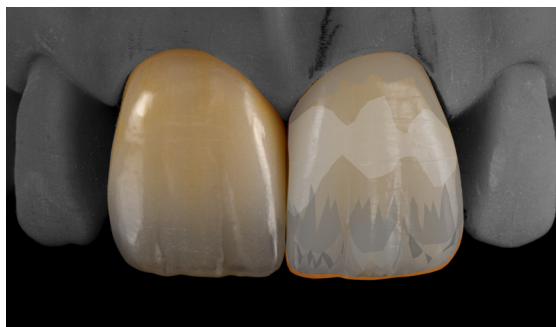


Step 9

Color/Mixture
Transpa Incisal 1
Ratio
Pure

🔥 Enamel/Correction firing

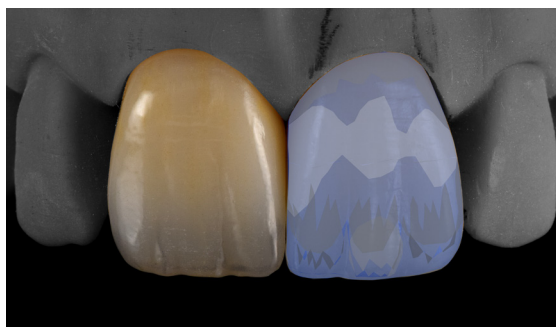

Final result

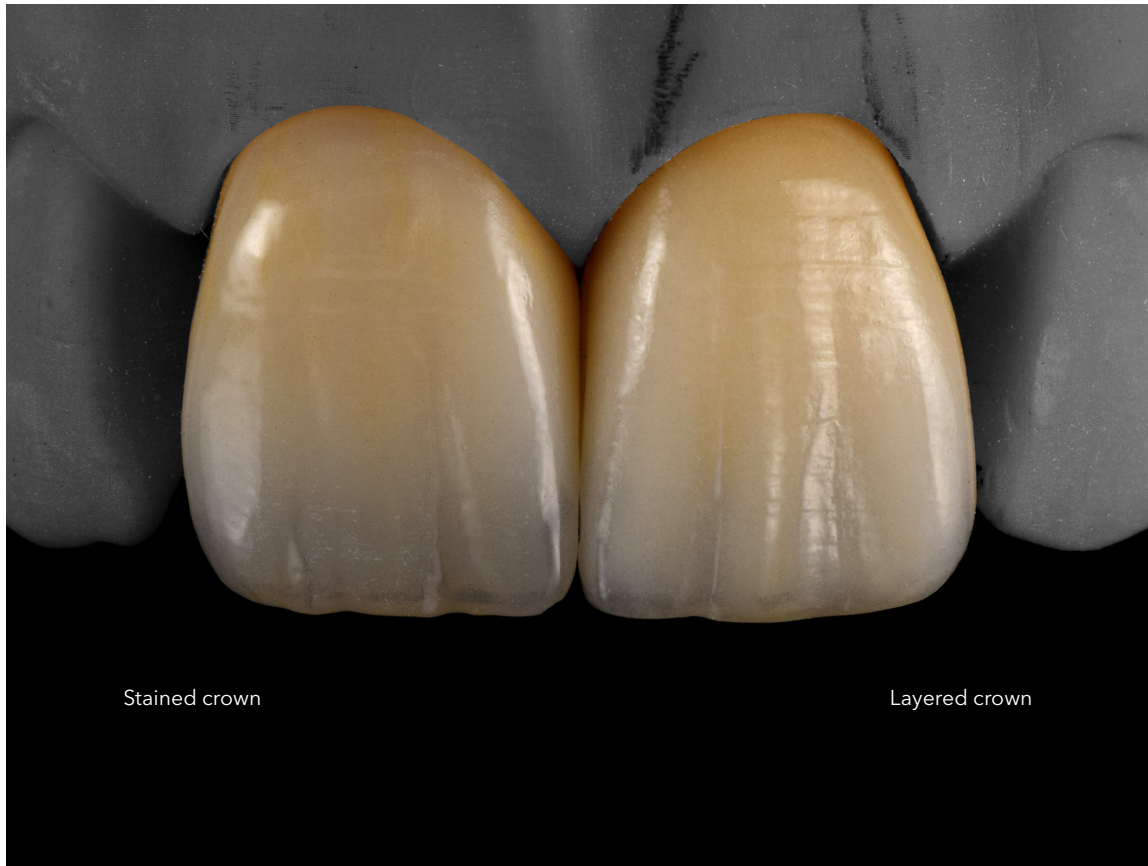



Stained crown

Layered crown



Dental designer

Ivan Unfeter
Dental Technician, Serbia

 dent.artist.unfeter
 dent.artist.unfeter

Contact us
info@sagemax.com

sagemax.com

Follow us
@sagemax.official

