

sagemax.com

sagemax®



Sagemax experience

PressCeramic
cut-back
by Florin Sabău

PressCeramic cut-back
by Florin Sabău

Sagemax experience

Initial situation & preparation



Discolored tooth and composite fillings.




Final preparation of teeth 12, 11, 21, 22.

Framework design & fabrication



Plaster Model.

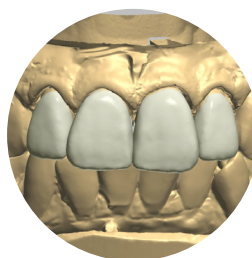

Framework design and color selection.




Pressed crowns out of PressCeramic MT B1.




Palatal silicone key.

Cut-back design & wash firing

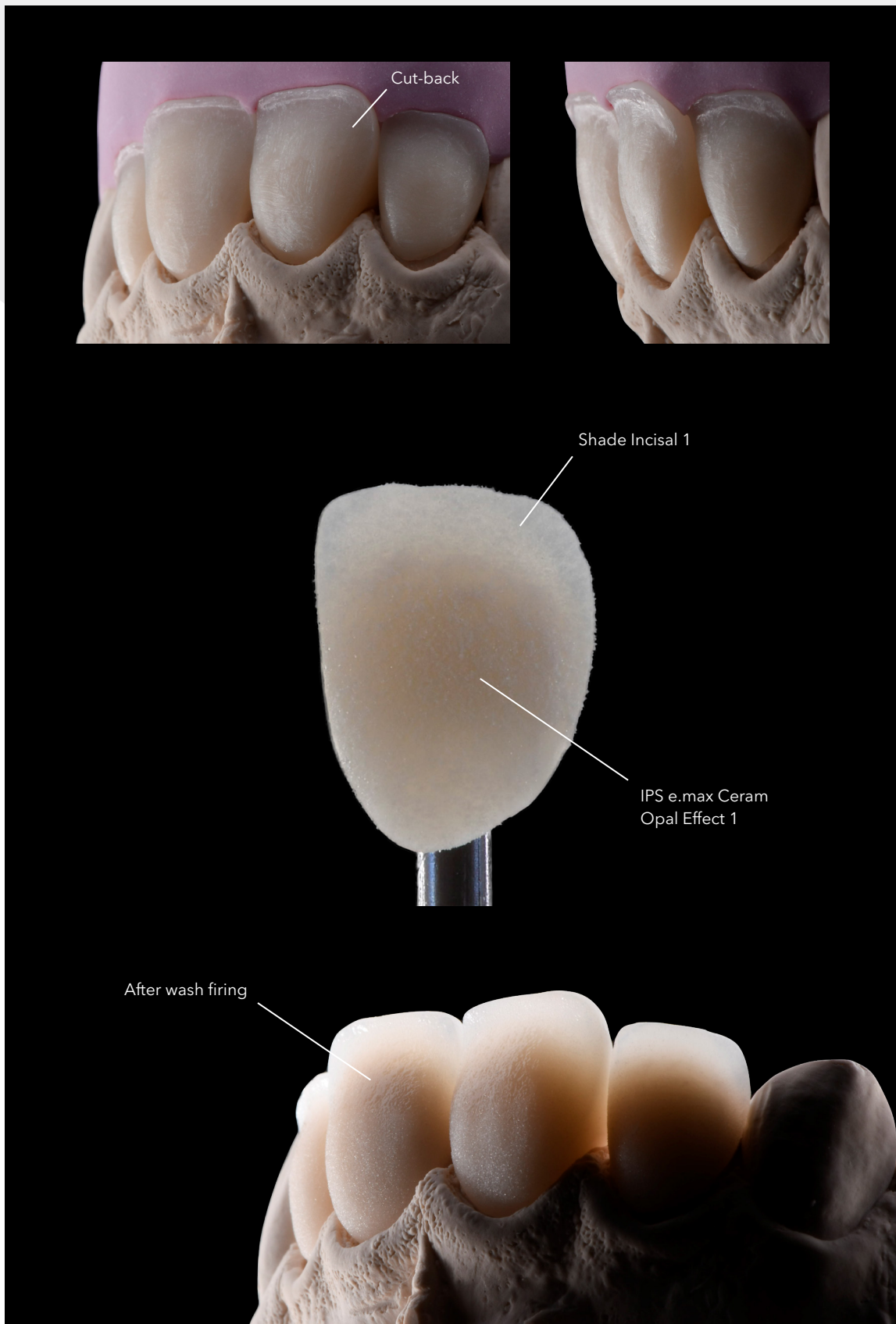


Cut-back design and characterization of the mesial and distal incisal areas using IPS Ivocolor Shade Incisal 1 (Co. Ivoclar), with IPS e.max Ceram Opal Effect 1 (Co. Ivoclar) sprinkled on top before firing.

Micro-layering & surface texture

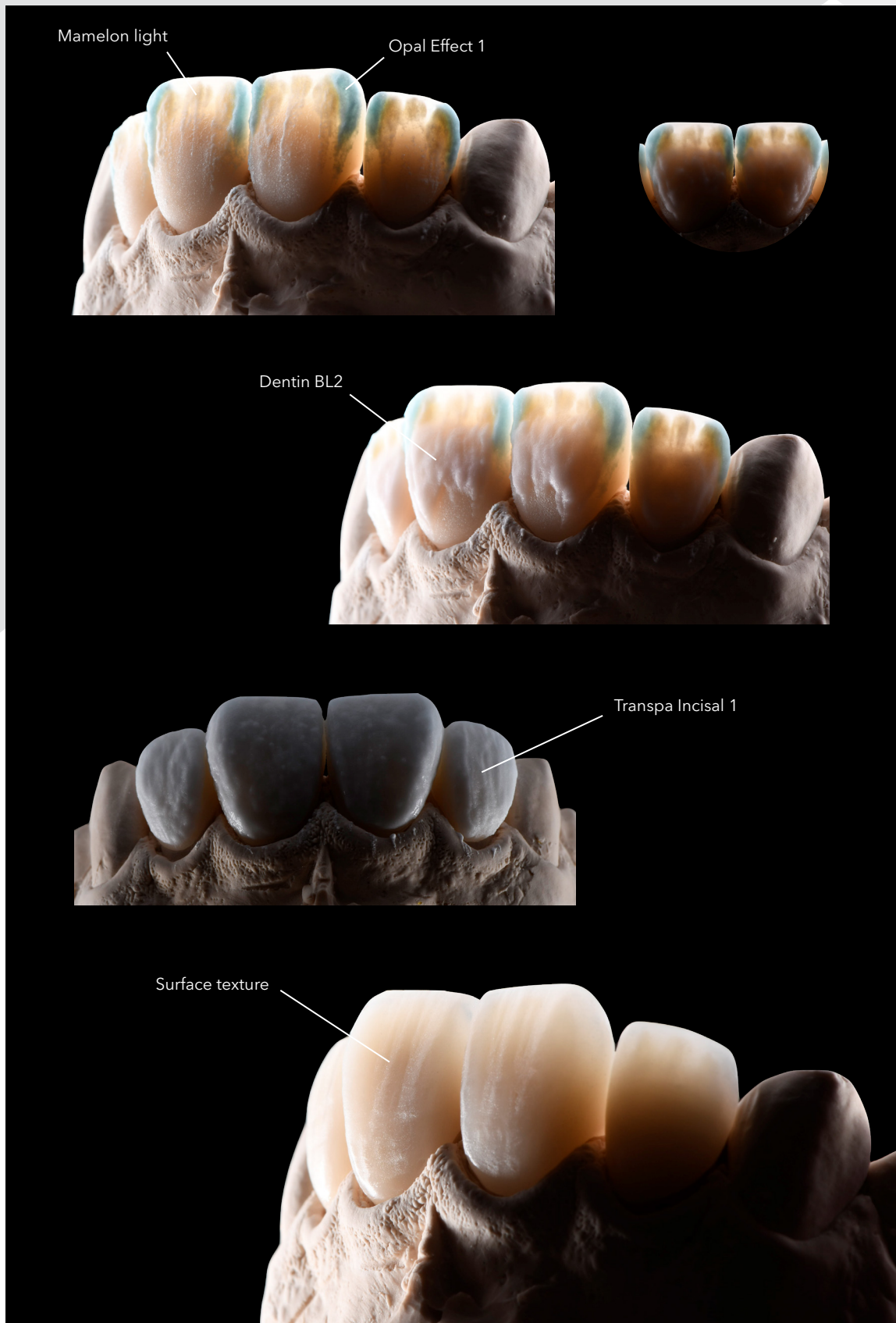

Layering IPS e.max Ceram Opal Effect 1 and IPS e.max Ceram Mamelon Light in the incisal area for a true-to-nature appearance. The tooth shape was finalized with a layer of IPS e.max Ceram Transpa Incisal 1.

Glazing & finalization

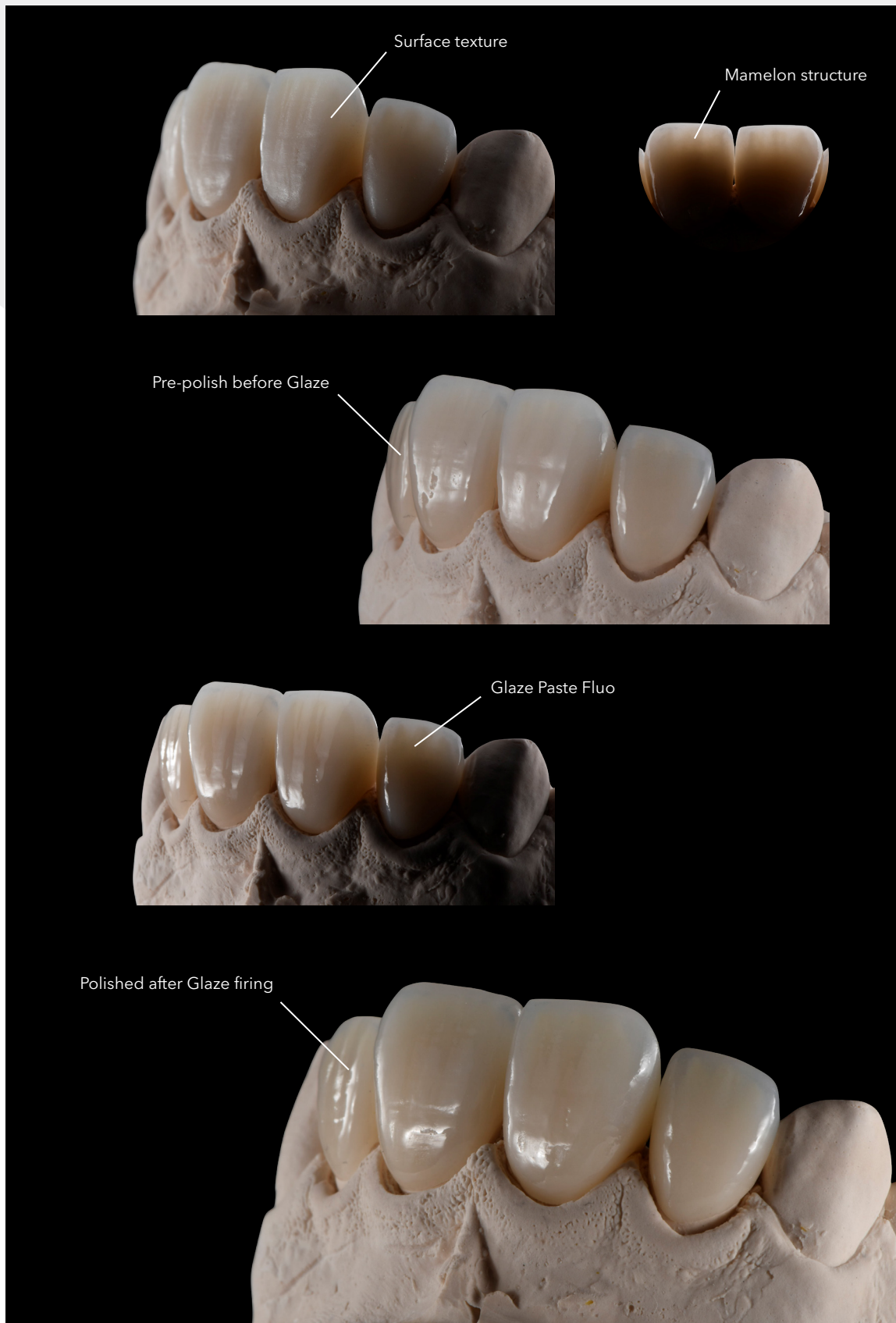

Pre-polishing of the surface, followed by glazing with IPS Ivocolor Glaze Paste Fluo. Finally, the surface is given a high-gloss polish.

Final result



The final result is truly impressive: the crowns look exceptionally natural and blend perfectly into the existing oral situation. The patient was overjoyed with the outcome.



Dental designer

Florin Sabău
Dental Technician, Romania

📷 florinsabau_

Contact us
info@sagemax.com

sagemax.com

Follow us
[@sagemax.official](https://www.instagram.com/sagemax.official)

