

sagemax.com

sagemax[®]



Sagemax experience

PressCeramic
micro-layered
by Dominik Monreal

PressCeramic micro-layered
by Dominik Monreal

Sagemax experience

Initial situation



Heavily discolored and deformed anterior teeth. Patient's request: masking of the anterior teeth with crowns.



After crown preparation of tooth 12 - 22.

Framework



Plaster model.




Wax copings.




Anatomical wax-up.



Micro cut-back.

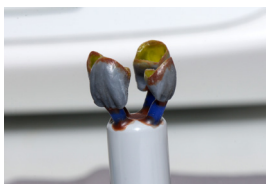

Sprued crowns.

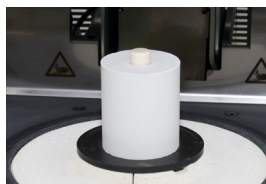

Pressing with PressCeramic
LT A3 pellet.



Fitted crown.



Prepared crowns.

Micro-layering



Wash firing with Transpa clear, "sprinkle technique".
Ceramic used: IPS e.max Ceram, Co. Ivoclar.



Individualization with Opal Effect 1, Dentin A3 and
Mamelon light.



Incisal build-up with Transpa Incisal 2.



Elaboration of the shape and surface structure.

Finishing



Staining and glazing with
IPS Ivocolor, Co. Ivoclar.



Crowns after stain and
glaze firing.



Etching with HF gel.



Finished crowns, ready for
cementing.

Final result



Before.



After.



The final result fits naturally into the surrounding oral cavity.

Dental designers



Dominik Monreal
Dental Technician, Germany

📷 dominikmonreal8



Kristina Günter
Dentist, Germany

www.zahnarzt-kleve.de
📘 Praxis für Zahnheilkunde Kristina Günter
📷 zahnarzt_kleve

Contact us
info@sagemax.com

sagemax.com

Follow us
@sagemax.official

